



# University of Minnesota

*Public Access Defibrillation*

## DEVICE USE



THIS DEVICE IS FOR USE ON ADULTS ONLY.

### To use this device:

- 1) Send someone to contact 911.
- 2) Remove device from cabinet and take to patient's side.
- 3) Open cover of device and turn on power.
- 4) Remove patches from cover and attach to patient's bare chest according to the diagram on the patches.
- 5) Connect patches to device.
- 6) Follow voice prompts—see inside cover of device for further instructions.

### Maintenance:

If cabinet or device is beeping, please contact police dispatch on their non-emergency number at 625-COPS. Give the dispatcher your name, location of the unit, and a description of the problem.